<b>3M ESPE</b>
<b>Cavit™</b>
<b>Cavit™ W</b>
<b>Cavit™ G</b>
<b><span>en</span></b> <b>Temporary Filling Material</b>
<b><span>ru</span></b> <b>Материал для временных пломб</b>
<b><span>bg</span></b> <b>Материал за временни запълнки</b>
<b><span>hr</span></b> <b>Materijal za privremene ispune</b>
<b><span>hu</span></b> <b>Ideiglenes tömőanyag</b>
<b><span>pl</span></b> <b>Materiał do czasowego wypełniania ubytku</b>
<b><span>ro</span></b> <b>Materiale pentru obturații provizorii</b>
<b><span>sk</span></b> <b>Materiál na dočasné výplne</b>
<b><span>sl</span></b> <b>Začasni polnilni material</b>
<b><span>cs</span></b> <b>Dočasný výplňový materiál</b>
<b><span>tr</span></b> <b>Geçici dolgu maddesi</b>
<b><span>et</span></b> <b>Ajutine täidismaterjal</b>
<b><span>lv</span></b> <b>Pagaidu plombas materiāli</b>
<b><span>it</span></b> <b>Laikinas užplidas</b>
<b><span>uk</span></b> <b>Тимчасовий пломбувальний матеріал</b>

Instructions for Use
Instrukcija po primenieniu
Указания за употреба
Upute za upotrebu
Használati utasítások
Instrukcja użycia
Instrucțiuni de Utilizare
Návod na použitie

**3M Deutschland GmbH**
Dental Products
Carl-Schurz-Str. 1
41453 Neuss – Germany

3M, ESPE and Cavit are trademarks of 3M or 3M Deutschland GmbH.
© 2020, 3M. All rights reserved.

<span></span>
<b><span>en</span></b> <b>ENGLISH</b>

**Product Description**
Cavit™ (pink), Cavit™ W (white) and Cavit™ G (grey) are radiopaque temporary filling materials self-curing under humidity for temporary filling of cavities.

Not all products are available in all countries.

☞ These Instructions for Use should be kept for the duration of product use.

**Indications**

- Temporary fillings

**Precautionary Measures**

3M MSDSs can be obtained from www.mmm.com or contact your local subsidiary.

**Preparation**

► In order to prevent tubes from leaking, use the tube key to press out the paste.

Application

► Use an instrument to fill the required quantity into the wet cavity. Cavit and Cavit W harden in 2 hours, Cavit G hardens in 4 hours. Avoid occlusal pressure during this time.

► Deep cavities need not be filled completely. If the material is used to close up a drug, pressure-free application is recommended.

► Use rotary instruments to remove the temporary filling.

**Note**

- Due to sensitivity to moisture, close the tube or jar immediately after removing the material, since otherwise the viscosity of the material increases (due to the entry of moisture), and the material may no longer be extractable.

**Storage and Shelf-life**

Store the product at 15–25° C/59–77° F. Store the product in a dry place.

Do not use after the expiration date.

Use open tubes and/or screw-cap jars as soon as possible.

**Customer Information**

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

**Warranty**

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user’s application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH’s sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

**Limitation of Liability**

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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