PL

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DENTIXPRO® - DISPOSABLE HEADREST COVERS

PRODUCT DESCRIPTION

DentixPro® disposable headrest covers can be used on dental chairs in dental surgeries, consulting, gynaecological, paediatric and physiotherapy and beauty rooms etc. **DentixPro® disposable headrest covers** are made of 2 ply tissue and 1 ply Poly film protection against moisture permeation and pathogenic microorganisms. The use of **DentixPro® disposable headrest covers** improves hygienic conditions during dental procedures and reduces likelihood of periprocedural infections at a moderate risk of contamination by fluids.

INTENDED USE

DentixPro® disposable headrest covers are non-sterile medical products allowing to maintain hygienic conditions to protect the patient against contaminants and disease causing infections during procedures carried on in dental surgeries, consulting, gynaecological and physiotherapy rooms.

DentixPro® disposable headrest covers are designed:

- 1) to protect the patient against contamination occurring during dental, medical and physiotherapeutic procedures;
- 2) to protect the patient against pathogenic infections that may occur during dental, medical and physiotherapeutic procedures.

DentixPro® disposable headrest covers are non-sterile and non-invasive disposable medical products.

DIRECTIONS FOR USE

Take the cover out from the packaging, put it on the chair headrest.

After a single use, put the cover in a plastic bag and follow applicable regulations on disposal of medical products.

CONTRAINDICATIONS

1. Do not use if you're allergic to polyester.
2. Do not use if allergic reactions, itching or other sensations and symptoms occur.
3. Do not apply to broken, infected or irritated areas of skin, undressed wounds, abrasions or ulcers.
4. Consult your dermatologist if symptoms of skin inflammation or hypertrophy occur.

CAUTION

1. Follow the instructions given on the packaging and instructions for use.
2. Product is designed for single use only. Dispose after use.
3. After use product should be treated as contaminated or potentially contaminated waste. To avoid infection, used product must be put in plastic bag and disposed of as medical waste in accordance with local requirements.
4. Store in a dry place. Do not expose to direct sunlight or sources of heat. Protect against damage, moisture, wet or contamination. Store in room temperature in dry and clean place.
5. Do not use in case of opened or damaged packaging.
6. Keep out of the reach of children.
7. Should not be used after the date printed on the packaging.

COMPOSITION:

Hygiene tissue (cellulose), polyethylene film (PE).

Number and date of issue: W&W/JSSK/Edition 1/Rev.0/01.2018