PL

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REF POCKET

PRODUCT DESCRIPTION

DentixPro® disposable dental bibs with pocket (MODEL: POCKET) can be used in dental surgeries, consulting, paediatric and physiotherapy rooms etc. They are also suitable for nursing and care services provided by care and curative centres, nurseries and kindergartens, or for home care services. **DentixPro® disposable dental bibs with pocket (MODEL: POCKET)** are made of 2 ply tissue efficiently absorbing physiological liquids (saliva) and pathological liquids (effusion, vomitus, purulent discharge, blood, etc.) and 1 ply Poly film protection against moisture permeation and pathogenic microorganisms. In the lower part disposable dental bibs have an integral pocket catching crumb during dental procedures. The use of **DentixPro® disposable dental bibs with pocket** improves hygienic conditions during dental procedures and reduces likelihood of periopercoludal infections at a moderate risk of contamination by fluids.

INTENDED USE

DentixPro® disposable dental bibs with pocket (MODEL: POCKET) are non-sterile medical products allowing to maintain hygienic conditions in order to protect the patient against contaminants and disease causing infections during dental procedures and are necessary to ensure that appropriate dental procedures are followed in dental surgeries, consulting rooms and physiotherapy rooms.

DentixPro® disposable dental bibs with pocket (MODEL: POCKET) are designed:

- to protect the patient against contamination occurring during dental procedures, such as saliva, blood, particles of dentine left after drilling, dental materials and drugs, etc;
 - to protect the patient against pathogenic infections that may occur during dental, medical and physiotherapeutic procedures;
 - as a barrier protecting the patient against liquid contaminants occurring during procedures, which are absorbed by the absorptive tissue and preventing pathogenic microorganisms, contamination and dust particles on the patients' clothes from getting into the oral cavity. The NORMAL model, which only consists of foil layer, does not have absorbent properties.
 - for protection of the oral cavity against pathogenic microorganisms and contamination from the patients' clothes during dental procedures;
 - to protect the patient against staining during feeding, in case of partially or completely disabled patients.
- DentixPro® disposable dental bibs with pocket** are non-sterile and non-invasive disposable medical products.

DIRECTIONS FOR USE

Take the bid out from the packaging, appropriately put it on the patient and fasten on the patient's neck by means of ribbons at the top. After a single use, put the bib in a plastic bag and follow applicable regulations on disposal of medical products.

CONTRAINDICATIONS

- Do not use if you're allergic to polyester.
- Do not use if allergic reactions, itching or other sensations and *symptoms occur*.
- Do not apply to broken, infected or irritated areas of skin, undressed wounds, abrasions or ulcers.
- Consult your dermatologist if symptoms of skin inflammation or hypertrophy occur.

CAUTION

- Follow the instructions given on the packaging and instructions for use.
- Product is designed for single use only. Dispose after use.
- After use product should be treated as contaminated or potentially contaminated waste. To avoid infection, used product must be put in plastic bag and disposed of as medical waste in accordance with local requirements.
- Store in a dry place. Do not expose to direct sunlight or sources of heat. Protect against damage, moisture, wet or contamination. Store in room temperature in dry and clean places.
- Do not use in case of damaged packaging.
- Keep out of the reach of children.
- Should not be used after the date printed on the packaging.

COMPOSITION:

Hygiene tissue (cellulose), polyethylene film (PE).

Number and date of issue: W8W/ISSK/Edtion 1/Rev.0/01.2018

	WEBER & WEBER Sp. z o.o. Pułców ul. Cieszyńska 229, 43-400 Cieszyń tel.: +48 338 529 100, fax: +48 338 529 200, www.weber.com.pl
	
	
	