**PRINT SPECIFICATION PAGE ONLY.  
DO NOT PRINT THIS PAGE IN THE FINAL ARTWORK.**

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2533111\_r00  
2020-03-25

### DURACROL™ Self-Curing Methacrylate Resin

#### DEVICE DESCRIPTION

Duracrol is a two-component methacrylate casting resin with a mineral filler, which is chemically cured without a heat supply.

Duracrol is available in either a jar or a bottle.

#### INDICATIONS FOR USE

Duracrol is indicated for manufacturing of custom trays and phantom models.

#### CONTRAINdications

The application of Duracrol is counter-indicated in the case of a known allergy to methyl methacrylate.

#### WARNINGS

- 1. Heating may cause a fire or explosion.
- 2. May cause an allergic skin reaction, respiratory irritation, skin, eye irritation
- 3. Highly flammable liquid and vapor.
- 4. Harmful if inhaled.

#### PRECAUTIONS

- 1. Avoid breathing in dust/fumes/gas (mist)/vapors/spray.
- 2. Contaminated work clothing should not be taken out of the workplace.
- 3. Avoid release into the environment.
- 4. Wear protective gloves/protective clothing/ eye protection/face protection.
- 5. If material remains in contact with the skin, wash with plenty of water.
- 6. Keep away from open flames, sparks, heat, hot surfaces.
- 7. Keep container tightly closed.
- 8. Store in a well-ventilated place.
- 9. This product is intended to be used by qualified dental professionals on a general patient population.

#### ADVERSE REACTIONS

Potential sensitivity to methyl methacrylate, especially in allergic patients.

#### INSTRUCTIONS FOR USE

The following methods are used for the manufacturing of a custom tray:

- Forming. The powder is mixed with the liquid in a 3:1 volume ratio. The plastic non-tacky substance is rolled (best results are achieved by inserting it between two cellulose sheets) to the desired thickness. It is adapted to the plaster model following the described procedure.
- Casting. A liquid mixture is poured onto a wet cellulose sheet, attached to a glass pane, which already has the desired shape, created, for instance, with a wet cord. When the dough ceases to stick, follow the procedure described above. The setting time is approx. 10–15 minutes.

#### CLEANING AND DISINFECTION INSTRUCTIONS

The product should be disinfected prior to use and after use on a patient with aqueous disinfection solutions. Refer to the disinfectant manufacturer's instructions for contact time and disinfectant concentration.

#### STORAGE, SHELF LIFE AND DISPOSAL

To be stored in a dry and dark place at 5–25 °C, in a well-sealed container. Refer to the outer packaging for the expiration date. Do not use after the expiration date. For proper disposal, always follow your local and national regulations.

#### VIGILANCE

If an adverse medical event occurs, report it to the manufacturer and to the competent authority for that country.

#### DISCLAIMER

The instructions provided above have been validated by SpofaDental as being CAPABLE of preparing a medical device, including reusable ones. It remains the responsibility of the dental professional / user to ensure that the reprocessing actually performed by equipment, materials and personnel is done per the instructions provided above in order to achieve the desired result. Any deviation by the dental professional / user from the instructions provided above will be done at your own risk; the SpofaDental will not be able to address any requests for refund or exchanges under guarantee of any products that have not been handled or reprocessed in accordance with the above instructions.

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