DE FD 370 cleaner Praxisreiniger

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EN FD 370 cleaner Surgery cleaning

FD 370 cleaner is a highly effective concentrate for the cleaning of disinfected surfaces such as floors, walls, practice fixtures, etc. Because FD 370 cleaner is compatible with almost all surface disinfectants, the disinfective action of disinfectants applied before or after the use of FD 370 cleaner is unaffected. FD 370 cleaner has an intensive cleaning action and is highly compatible with other materials. There is no stickiness or discoloration from incompatibility reactions.

Composition: FD 370 cleaner contains non-ionic tensides (5 – 15 %), complexing agents (< 5 %) and auxiliary substances in an aqueous solution.

Directions for use: The working concentration should be 1 – 2 % depending on the degree of contamination. For example, add 80 ml or 160 ml using the measuring cup in the bottle cap (the mark represents 20 ml), then add lukewarm water to make 8 litres of solution. Wipe floors and surfaces until moist. Ensure that the surfaces being cleaned are evenly wet.

Use FD 370 cleaner separately from disinfection. Do not mix with the disinfectant! Do not mix FD 370 cleaner with commercial domestic cleaning agents. We recommend using a suitable wet-wipe procedure.

Caution: Commercially available domestic cleaners generally contain anionic tensides, which are not compatible with some of the active ingredients in surface disinfectants. FD 370 cleaner avoids this problem associated with previous cleaners as it was developed specifically to clean disinfected surfaces.

see imprint.

Warning. Causes serious eye irritation. Wear protective gloves and eye/face protection. IF SWALLOWED: rinse mouth. Do NOT induce vomiting. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Store in a well-ventilated place. Keep container tightly closed. Dispose of contents/container to hazardous or special waste collection point.

DÜRR DENTAL (Produkts) UK Ltd. 14 Linnwell Way - Telford Way Industrial Estate Kettering, Northants NN16 8PS, United Kingdom ☎ +44 1536526740

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