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SOLO PER USO PROFESSIONALE

EN - INSTRUCTIONS FOR USE: Disinfectant for aspiration circuits. Concentrated aldehyde and phenol free liquid for disinfecting, cleaning and deodorising aspiration systems and spittoons.

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EFFECTOS COLATERALES: No se han observado efectos colaterales.

INSTRUCCIONES PASO A PASO:

- Rinse the aspiration system by drawing up 2 litres of cold water (fig. 1).
- Prepare at least one litre of solution for each dental unit; fill a recipient with water and then add 10 ml of Zeta 5 Power Act for each litre of water (fig. 2, 3).
- The solution will be opaque.
- Suck in the solution through the aspiration tubes (fig. 6). Pour at least 250 - 300 ml of disinfectant solution into the spittoon.
- Leave to work for 15 minutes for a rapid disinfection. Leave to work for at least 4 hours (overnight) for the spectrum of action indicated in the table (inc. tuberculicidal).
- At the end of the time of action, rinse the aspiration system by sucking in 2 litres of cold water (fig. 1).

Repeat this practice at least at the end of each working day but preferably also in the middle of the day and after a particularly invasive operation.

SPECTRUM OF ACTION

	Reference standard	Concentration	Time of action (minutes)
Efficacy (microorganisms)			
Bactericidal (S. aureus, P. aeruginosa, E. hirae)	EN13727	1%	15
	EN14561	1%	15
Bactericidal (S. aureus, P. aeruginosa, E. hirae)	EN13624	1%	15
	EN14562	1%	15
Yeasticidal (C. albicans)	EN14562	1%	15
Tuberculicidal (M. terrae)	EN14348	1%	240
	DVV/RKI	1%	1
Limited virucidal effect towards encapsulated and lipophilic viruses including blood viruses (HIV, HBV, HCV), Herpes simplex and families of virus such as the Orthomyxoviridae (including the gripe human and animal influenza viruses such as H1N1 and H1N1), flaviviridae (virus della Ebola) and paramyxo- viruses (measles virus)	DVV (non-porous surfaces)	1%	60
	prEN16777	1%	60

IMPORTANT NOTES: The instructions must be respected, irrespective of possible information given in any other way, including during

demonstrations. It is the operator's responsibility to ensure that the product is suitable for the application in question. The Company is not liable for possible damage, including third parties, deriving from failure to respect the instructions or inappropriate use of the product.

The company does not bear responsibility for damages, including third parties, which may result from the use of the products supplied.

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WARNINGS: Causes serious skin burns and eye damage. In the event of contact with the skin or hair, remove the contaminated garments and rinse the skin. In the event of contact with eyes, rinse thoroughly for several minutes. Remove contact lenses if present and easy to do. If the irritation persists, consult a doctor. If swallowed, rinse the mouth, do not induce vomiting and contact a poison centre or doctor immediately.

PRECAUCIONES: Wear protective gloves and garments and protect the eyes and face. Wash your hands after handling. Do not dispose of it in the environment; dispose in accordance with local laws and regulations. Keep the product in a tightly closed container in a cool dry place out of direct sunlight. The shelf life of the product is valid only if unopened and correctly stored. Keep the product at a maximum temperature of 27°C.

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