## GC Fuji™ I CAPSULE

RADIOPAQUE GLASS IONOMER LUTING CEMENT IN CAPSULES

For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS

- Cementation of metal-based inlays, onlays, crowns, bridges and posts
- Cementation of high strength (zirconia based) all ceramic crowns and bridges

### CONTRAINDICATIONS

- Pulp capping
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

### DIRECTIONS FOR USE

Powder / Liquid ratio (g/g)	0.33/0.18
Mixing time (sec.)	10"
Working time (min., sec.) (at 23 °C) (from start of mixing)	2'15"
Final finishing commencing time (after sealing the restoration)	4'30"

### 1. TOOTH PREPARATION

- Prepare tooth in usual manner. For pulp capping, use calcium hydroxide.
- Clean the prepared teeth with pumice and water.
- Rinse thoroughly with water. Remove excess moisture by blotting with a cotton pellet or gently blowing with an oil-free air. DO NOT DESICCATE. Prepared surfaces should appear moist (glistening).

### 2. RESTORATION PREPARATION

Make sure that the restoration is pretreated and handied according to the manufacturer's instructions.

### 3. CAPSULE ACTIVATION AND MIXING

- Before activation, shake the capsule or tap its side on a hard surface to loosen the powder (Fig. 1).
- To activate the capsule push the plunger until it is flush with the main body (Fig. 2).
- Immediately place the capsule into a GC CAPSULE APPLIER and click the lever once (Fig. 3). The capsule is now activated.

Note : The capsule should be activated just before mixing and used immediately.

- Immediately remove the capsule and set it into a mixer (or an amalgamator) and mix for 10 seconds (+/-4.000 RPM) (Fig. 4).

Note : When using a rotational mixing device, it is recommended not to use the centrifuge option.

### 4. DISPENSING

- Immediately remove the mixed capsule from the mixer and load it into the GC CAPSULE APPLIER.
- Make two clicks to prime the capsule (Fig. 5). To adjust the direction of the nozzle, hold the applicator with the capsule towards you and turn the capsule body.

- Extrude directly into the restoration.
- To remove the used capsule, push the applicator release button. Twist the end and pull upwards.

### 5. CEMENTATION

- Coat the internal surface of the restoration with sufficient cement and seat immediately. The working time is 2 minutes 15 seconds from start of mixing at 23°C (73.4°F). Higher temperatures will shorten working time.
- Maintain moderate pressure.
- Start removing excess cement when the excess cement feels rubbery.
- Finishing can be started 4 minutes 30 seconds after sealing the restoration.

### STORAGE

Store the capsules in the original aluminium foil in a cool and dark place (4-25°C) (39.2-77.0°F). (Shelf life : 2 years from date of manufacture).

### SHADE

Light yellow.

### PACKAGE

50 capsules  
Average contents per capsule : 0.33g powder and 0.18g (0.14mL) liquid.  
Minimum net volume of mixed cement per capsule : 0.19mL.  
Option : GC CAPSULE APPLIER (1 piece).

### CAUTION

- In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water. To avoid contact, a rubber dam and/or cocoa butter can be used to isolate the operation field from the skin or oral tissue.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- This product is not indicated for filling or core build-up.

Last revised : 12/2014

MANUFACTURED BY  
GC CORPORATION  
76-1 Hiasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU: GC EUROPE N.V.  
Distributed by Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33, B-3001 Leuven, Belgium  
TEL : +32 16 74 10 00  
PRINTED IN BELGIUM

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