PL

Eucalyptol

Olejek do rozpuszczania gutta-perchi

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Eucalyptol

Oil for dissolving gutta-percha

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY

COMPOSITION: eucalyptus oil 100%

INTENDED USE

EUCALYPTOL is intended for use during repeated endodontic treatments to remove gutta-percha from the root canal.

INSTRUCTIONS FOR USE

Before the treatment, immerse the attached pipette into ethyl alcohol.

- Using the pipette, insert 1-2 drops of liquid into the root canal.
- Remove gutta-percha.
- Use another dose of preparation if necessary.
- If gutta-percha is still dry or crumbly, apply EUCALYPTOL again.
- Repeat long as needed.

The product is reusable while the pipette is disposable. Multiple use of the pipette may cause the risk of re-infection.

CONTRAINDICATIONS

Do not use for patients who are hypersensitive to the product components.

PRECAUTIONS

It can cause allergic skin reaction. Use rubber-dam, gloves, protective glasses and protective clothing while working with the product.

STORAGE

Store in original packaging below 25 °C.

Keep out of reach of children.

Expiry date is placed on the main container. Once container is open, the expiry date does not change if the container is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING

Bottle with 10 ml of preparation, application pipettes.

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