PL

# ALUSTAT

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# ALUSTAT

Liquid for staunching bleeding

**BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY**  
PRODUCT FOR DENTAL USE ONLY

**COMPOSITION:** aluminium chloride 25%, water Ph. Eur., menthol, excipients.

#### INTENDED USE

ALUSTAT is intended for use during dental treatment to staunch slight gingival bleedings. Aluminium chloride in the product has the vasoconstrictive effect on tiny blood vessels, as a result of which the gingival swelling and slight bleedings decrease.

#### INSTRUCTIONS FOR USE

Rub gently the swollen or injured gingiva with a cotton wool tampon soaked in ALUSTAT and rinse with clean water. Repeat if necessary. Recommended contact time up to 5 minutes.

#### CONTRAINDICATIONS

Do not use for patients who are hypersensitive to the product components.

#### PRECAUTIONS

Irritating to the mucous membranes of the oral cavity, skin and eyes. In case of contact, rinse immediately with plenty of water and seek medical advice. If swallowed, do not induce vomiting. Drink plenty of water. Seek medical advice. Use disposable gloves and protective glasses while working with the product.

#### STORAGE

Store in original packaging below 25 °C. Keep out of reach of children. Expiry date is placed on the main container. Once container is open, the expiry date does not change if the container is securely closed after each use.

#### HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

#### PACKAGING

Bottle with 10 g of product and dropper.

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